PALMARE

WINKELSTABILE RADIUSPLATTE

Palmar fixed-angle radial plate

- Titan

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The new 2 mm palmar radius plate is an implant precisely designed for the anatomical conditions of the palmar distal end of the radius. The 6 distal 2 mm screws are fastened to the plate with angular stability and support the joint area in an anatomically aligned gallery arrangement. Small fragments supporting the joint area can also be securely fixed in this way.

The slot is in the proximal shaft region of the plate. It ensures that the plate is positioned correctly in the shaft region after the fine repositioning of the plate.

The mechanical strength of the two distal shaft screws, which are subject to the greatest forces, is significantly higher, because they are only rotated once.

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- Smith fractures, reverse Barton fractures (23 – B3)
- Distal radius extension fractures with an without involvement of the joint (23 – A3 / C1/ C2 /C3)
with preference for palmar access.

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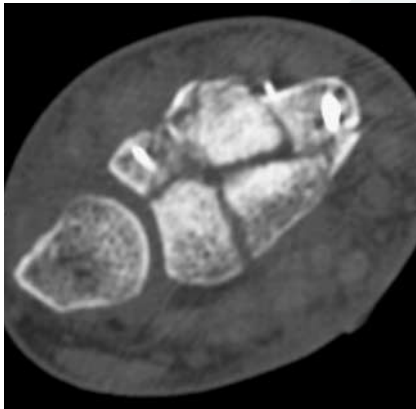
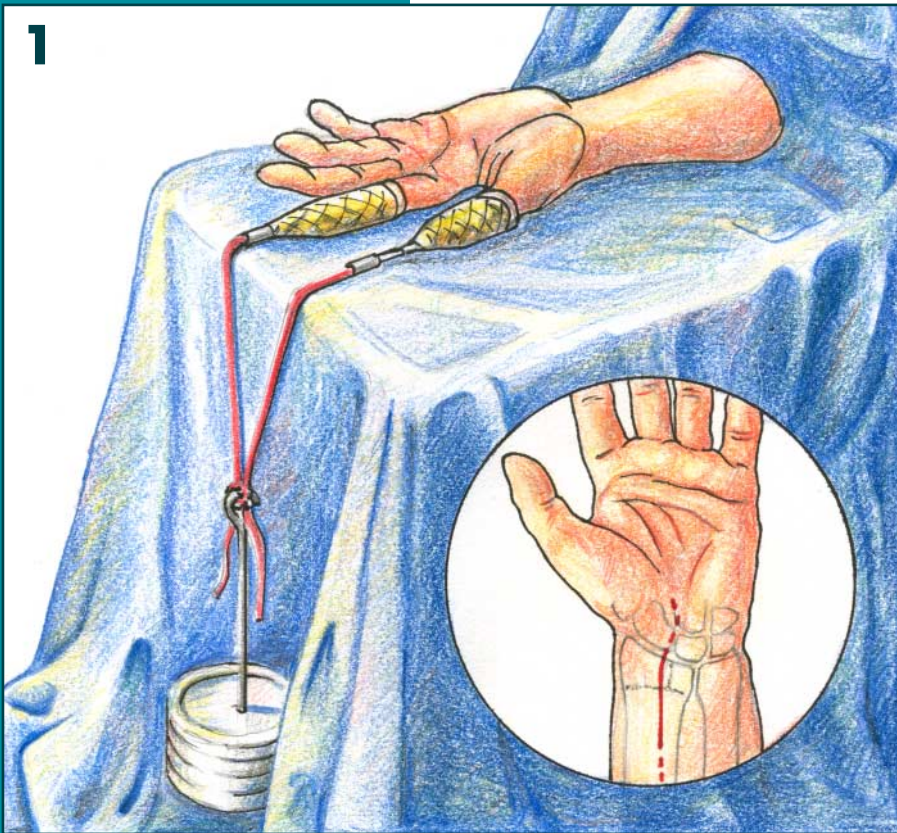
Postoperative brief immobilisation in a plaster splint, immediate exercises, ergotherapy etc. in the splint.

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
Supine position, hand table, bloodless field, extension using traction extension sleeves on the first and second digits (3-4 kg). Sterile cover. 7 cm long skin incision radial from the flexor carpi radialis tendon.

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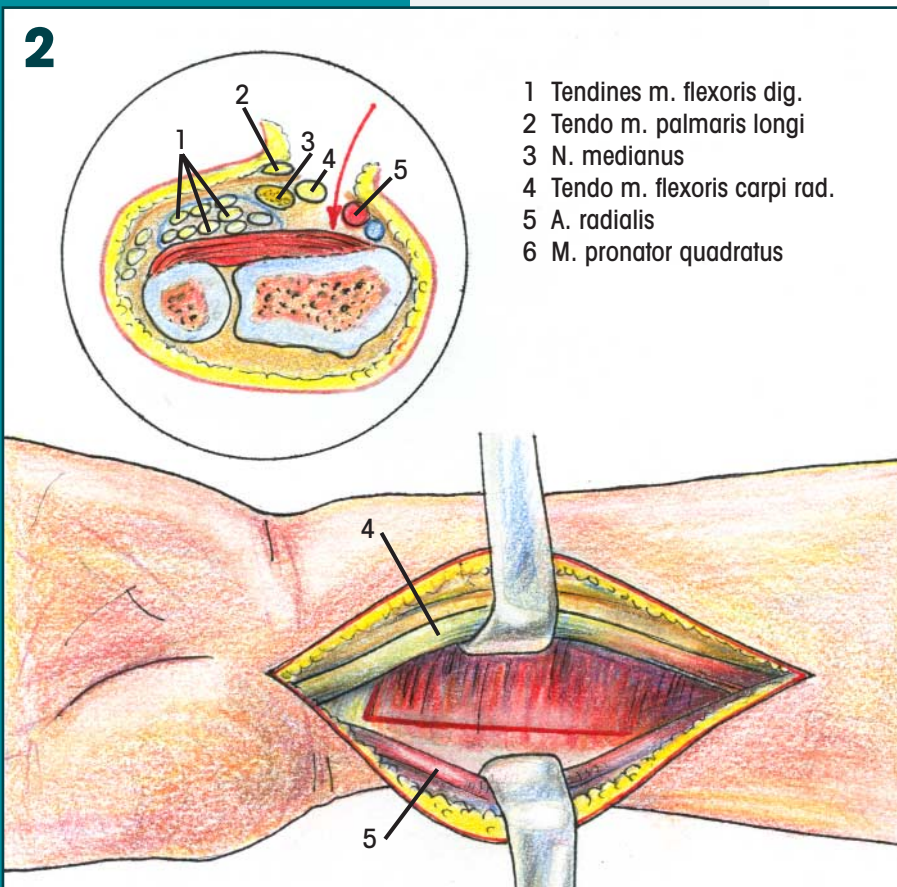Preparation between flexor carpi radialis tendon and radial artery. L-shaped removal of the pronator quadratus. Exposure of fracture.

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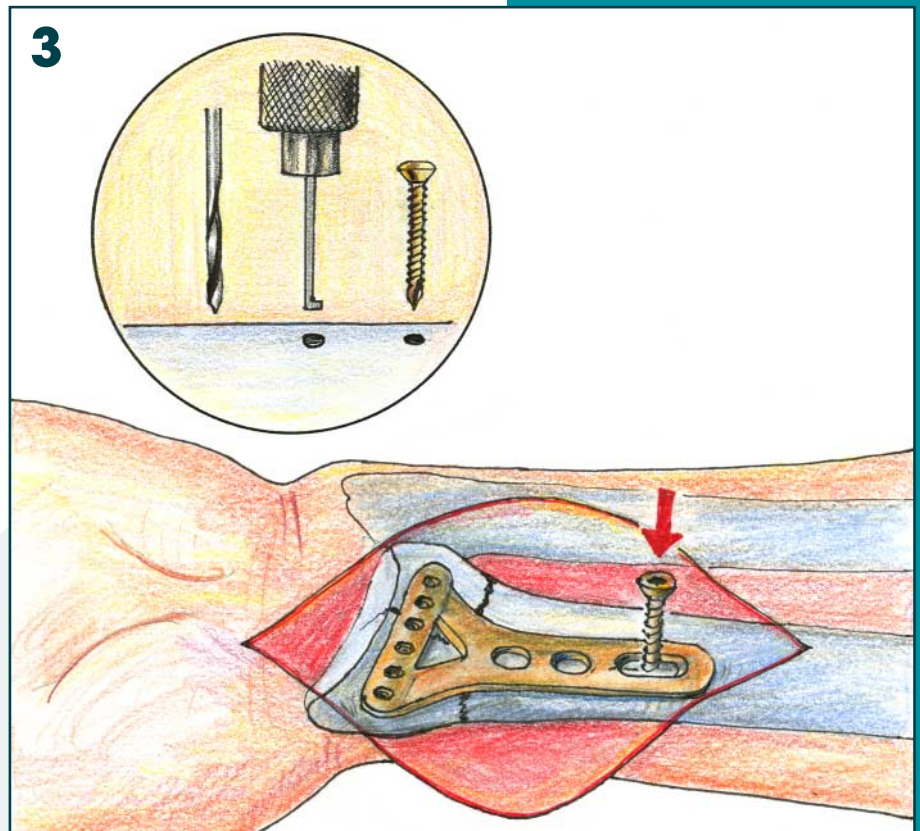
Fine repositioning of the fracture. Fit plate and insert 3.5 mm cortical screw in centre of slot. Fine correction of plate position, other shaft holes covered.

When restoring extension fractures the distal plate holes can also be first fixed with 2 mm screws for angular stability with a plate raised from the shaft. Then the repositioning is carried out as a whole by raising the plate at the radius shaft.

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
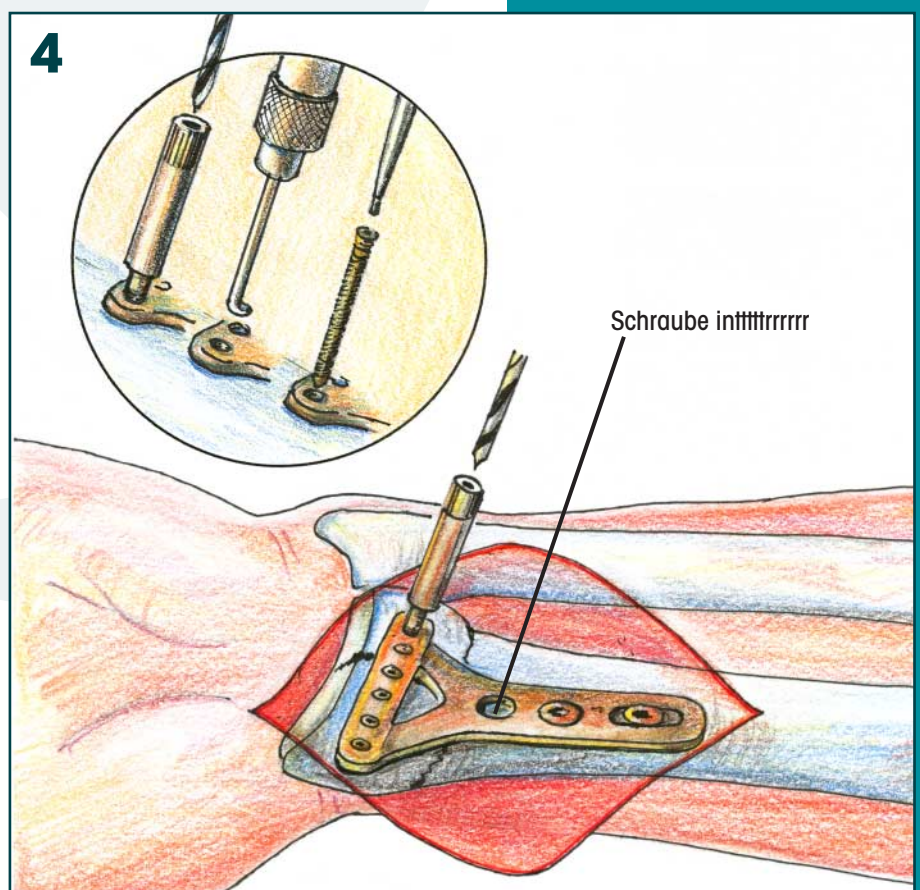
Screwing in the 2 mm drill sleeve, pilot drilling, removal of the drill sleeve, determining the length, screwing in the 2 mm screws for angular stability. They must not protrude above the dorsal cortex.

As many screw holes should be used as required by the fracture with its fragments. Final check under BW and documentation of the osteosynthesis with printer. Open the blood block, Redon drainage, close wound.

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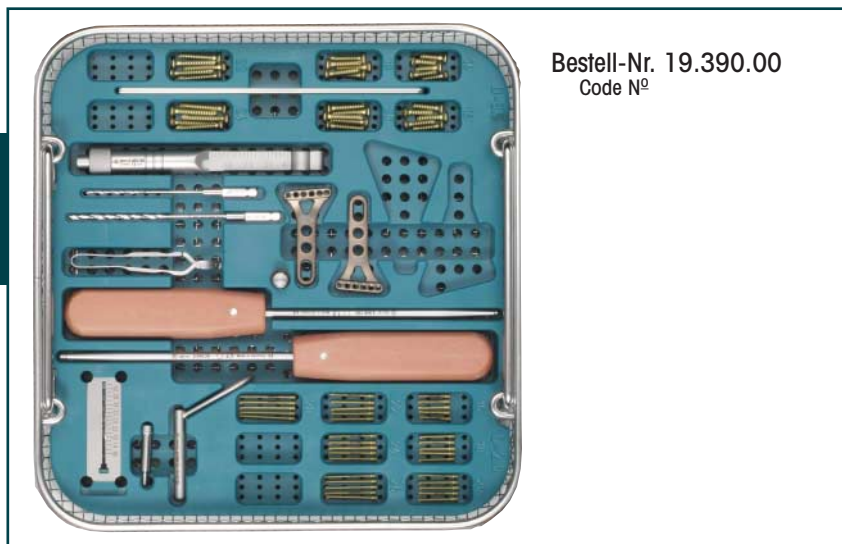
Cortical screws titan, D 2,0 mm (conical head thread, self-tapping)

Länge	Length	Anzahl	Quantity in set
16 mm		5	
18 mm		5	
20 mm		5	
22 mm		5	
24 mm		5	
26 mm		5	
28 mm		5	

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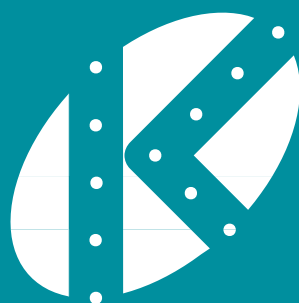Radial plate palmar

Winkelstabil, Titan, rechts / Fixed-angle, titan, right

Winkelstabil, Titan, links / Fixed-angle, titan, left

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Screws titan, D 2,0 mm (conical head thread, self-trapping)

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Cortical screws titan, D 3,5 mm (self-trapping)

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Locating sieve (for instruments and implants)

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2.953.50

Kirschnerdraht Trokar, Ende rund D 1,4 mm

Kirschner-wire with trokar point, rounded end D 1,4 mm

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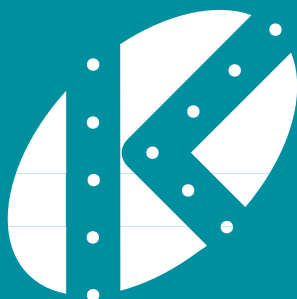
No part of this publication may be reproduced, stored in a retrieval system or transmitted in any form or by any means electronic, mechanical, photocopying, recording or otherwise without the prior permission of the publisher.

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OSTEOSYNTHESE GmbH
KÖNIGSEE IMPLANTATE UND INSTRUMENTE ZUR OSTEOSYNTHESE GmbH